


Cleaning and Storing the Mucosal Atomization Device (MAD) Nasal Atomizer

Single person use only.²

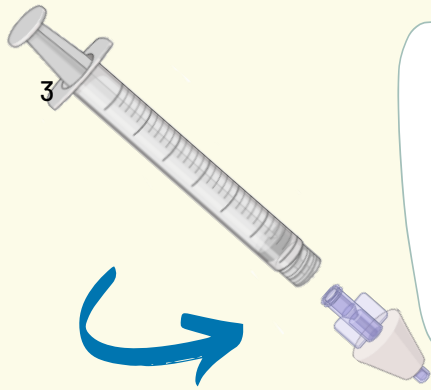

1

Remove atomizer from syringe.

2

Wash the MAD with mild soap and water.



3

Rinse Well




4

Use syringe to rinse the MAD and let air dry.



5

Place in a safe storage container once dry.

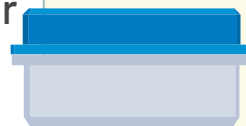

Replace after 3-4 uses.¹

Est Oct 2022 Nursing Support Services



¹ Bahlibi, A. (2021, May 25). Hint: How to use a nasal atomizer to safely deliver intranasal medication. Connected Care @ SickKids. Retrieved September 27, 2022, from <https://www.connectedcare.sickkids.ca/quick-hits/nasalatomizer>

² MAD Nasal, Morrisville, Teleflex Medical, 2017

³ Nasal atomizer and syringe image, Created with BioRender.com.

MAD Nasal™

REF MAD100
MAD110
MAD300

en	INTRANASAL MUCOSAL ATOMIZATION DEVICE	Rx ONLY
de	INTRANASALER MUKOSALER ZERSTÄUBER	 
fr	DISPOSITIF DE PULVÉRISATION DE MÉDICAMENT SUR LA MUQUEUSE NASALE	
es	DISPOSITIVO DE ATOMIZACIÓN MUCOSA INTRANASAL	
it	ATOMIZZATORE INTRANASALE PER VIA MUCOSALE	
nl	INTRANASALE VERNEVELAAR VOOR DE SLIJMVLIEZEN	
sl	PRIPOMOČEK ZA AEROSOLIZACIJO ZA INTRANAZALNO SLUZNICO	
cs	INTRANAZÁLNÍ SLIZNIČNÍ ATOMIZAČNÍ ZAŘÍZENÍ	
pt	DISPOSITIVO DE ATOMIZAÇÃO DA MUCOSA INTRANASAL	
da	NÆSESLIMHINDESPRAY	
fi	INTRANASAALINEN LIMAKALVOJEN SUMUTUSLAITE	
el	ΣΥΣΚΕΥΗ ΨΕΚΑΣΜΟΥ ΕΝΔΟΠΡΙΝΙΚΩΝ ΒΛΕΝΝΟΓΟΝΩΝ	
no	INTRANASAL MUKOSAL FORSTØVERENHET	
pl	WEWNAŹRZOSOWY ATOMIZATOR ŚLIZÓWKOWY	
ro	DISPOZITIV PENTRU PULVERIZARE PE MUCOASA NAZALĂ	
sk	INTRANAZÁLNE SLIZNIČNÉ ATOMIZAČNÉ ZARIADENIE	
sv	INTRANASAL MUKOSAL ATOMISERINGSANORDNING	
tr	İNTRANAZAL MUKOZAL ATOMİZASYON AYGITI	
uk	ІНТРАНАЗАЛЬНИЙ РОЗПИЛЮВАЧ ДЛЯ ЗРОШЕННЯ СЛИЗОВОЇ ОБОЛОНКИ	
ja	耳鼻咽喉科用薬液噴霧器 (鼻腔内粘膜用)	
zh_cn	鼻粘膜雾化装置	

en

INTRANASAL MUCOSAL ATOMIZATION DEVICE

INDICATIONS FOR USE: For atomizing topical solutions across the naso and oropharyngeal mucous membranes.

INSTRUCTIONS FOR USE:

1. Disconnect MAD Nasal™ device from included syringe (MAD100, MAD110 only).
2. Fill syringe with desired volume of solution and eliminate remaining air.
3. Connect the MAD Nasal device to the syringe.
4. Place the MAD Nasal device tip in the nostril or oropharyngeal cavity.
5. Compress the syringe plunger to spray atomized solution into the nasal or oropharyngeal cavity.
6. Repeat in other nostril if indicated.

WARNING: Single patient use: The use of products intended for single patient use on more than one patient creates a potential risk of serious injury and/or infection which may lead to death. Reprocessing of medical devices intended for single patient use only may result in degraded performance or a loss of functionality.

SPECIFICATIONS:	
TYPICAL PARTICLE SIZE	30-100 microns.
SYSTEM DEAD SPACE	MAD100 = 0.15mL MAD110 = 0.15mL MAD300 = 0.06mL
TIP DIAMETER	0.17 inches (4.3 mm)
APPLICATOR LENGTH	1.69 inches (4.3cm)
SYRINGE ACCURACY (MAD100 & MAD110)	+/- 5%

For use with drugs approved for intranasal delivery

MAD Nasal is a trademark of Teleflex Incorporated or its affiliates.